STEROS ACTIVE SPRAY

Class IIb
medical device
CE 0546
REF SA0.50-10
Rev. 0_12/01/2026

FREE RADICALS STERILIZATION SYSTEM

Fast sterilizing for Medical Devices and surfaces of Medical Devices

icm S.r.l. SRN: IT-MF-000044906
Contrada Torre, snc | 89040 Portigliola (RC) | Italy
Tel.-Fax +39 0964 361060
info@icmchemical.it | icmchemical.it

LAMBDA S.p.A.
Via dell'Impresa | 36040 Brendola (VI) | Italy
T +39 0444 349165 | F +39 0444 349954
info@lambdaspa.com | lambdaspa.com

DESCRIPTION AND FIELDS OF APPLICATION

STEROS ACTIVE SPRAY is a cold free radical-generating powder sterilant specifically designed for the sterilization of invasive and non-invasive medical devices, surfaces of medical devices such as operating tables, dental units, instrument shelves, and healthcare equipment. It sterilizes in a few seconds, preventing the risks of infection and ensuring safe protection for operators and patients.

PACKAGE CONTENTS

- Code SA0.50-10: 10 vials each containing 2.25 g of powder.
- Each bottle allows the preparation of 500 ml of active solution.
 - Each package includes one 500 ml dispenser with spray dispenser for preparing the active solution.

COMPOSITION

Sulphated peroxides - sodium bromide buffered with alkaline carbonates.

API

Free radicals with reducing activity in aqueous solution at basic pH.

MODE OF ACTION

Transfer of electrons that react with the structure of microorganisms causing irreversible chemical damage and their immediate destruction.

ACTIVITY

Effective against Gram+ and Gram- bacteria | viruses | mycobacterium tuberculosis | fungi | spores

CONTACT TIME 15 seconds

POWDER SHELF LIFE 24 months

ACTIVE SOLUTION SHELF LIFE 7 days

STORAGE CONDITIONS Store between 5 °C and 40 °C

FEATURES AND ADVANTAGES

- Sterilizes in a few seconds; prevents the risk of infection as it rapidly inactivates microorganisms and prevents their migration..
- The basic pH values and the reducing activity of radicals stopping oxidation phenomena and do not compromise the integrity of surfaces and instruments; the solution does not stain and does not leave streaks.
- The single-dose vial facilitates the preparation of the active solution; the powder is dissolved directly in the bottle equipped with a spray dispenser.
- The powder dissolves in tap water at room temperature. The ready-to-use solution assumes a light yellow color that confirms to the operator the start of the sterilizing activity.
- The solution does not emit dangerous vapors into the work environment.
- Accidental contact of the solution with the skin does not cause irritation (the powder may cause irritation if the skin is wet).
- The exhausted solution complies with legal discharge limits and may be disposed of in the sewage network.
- The small size package avoid any space issues, ensuring easy transport and storage.

The cold sterilization system with free radicals is patented.

DOSE OF USE 1 single-dose vial of 2.25 g for 500 ml of water (0.45%)

USE

Preparation of the solution:

- Fill the 500 ml bottle B with water.
- Transfer the powder contained in vial A into the 500 ml bottle B previously filled with water.
- Close bottle B with the provided spray dispenser and shake well to help the dissolution of the powder.
- Wait about 10 minutes.
- The solution will be clear and take on the characteristic light yellow color that confirms the start of the sterilizing activity. If the tap water has a hardness level higher than 20°F the solution may appear cloudy.
- The obtained solution is active and ready to use, and is valid for 7 days from preparation.

Sterilization of medical devices and surfaces of medical devices:

- Spray the solution abundantly.
Avoid spraying the product on surfaces with traces of other disinfectants, detergents, surfactants or acidic substances.
- Wait for the activity time (15 seconds) and dry.

NB: Before use, once the active solution has been prepared, write the following data in the appropriate spaces on the 500 ml bottle B:

- Batch number of the powder product reported on vial A.
- Expiry date of the active and ready to use solution (7 days from preparation).

TRACEABILITY OF THE STERILIZATION PROCESS

After preparing the sterilizing solution, the label can be removed from the empty vial and applied on the appropriate register to allow traceability of the sterilization process.
The product is intended for use by healthcare professionals.

PRECAUTIONS

H302	Harmful if swallowed
H314	Causes severe skin burns and eye damage
H317	May cause an allergic skin reaction
H318	Causes serious eye damage
H334	May cause allergy or asthma symptoms or breathing difficulties if inhaled
H412	Harmful to aquatic life with long lasting effects

PRECAUTIONARY STATEMENTS

Prevention

P260	Do not breathe dust/fume/gas/mist/vapours/spray
P261	Avoid breathing dust/fume/gas/mist/vapours/spray
P264	Wash thoroughly after handling
P270	Do not eat, drink, or smoke when using this product
P272	Contaminated work clothing should not be allowed out of the workplace
P273	Avoid release to the environment
P280	Wear protective gloves/protective clothing/eye protection/face protection
P285	Wear respiratory protection if required

Response

P301+P312	IF SWALLOWED: call a POISON CENTER/doctor/... if you feel unwell
P301+P330+P331	IF SWALLOWED: rinse mouth. Do NOT induce vomiting
P302+P352	IF ON SKIN: wash with plenty of water/...
P303+P361+P353	IF ON SKIN (or hair): take off immediately all contaminated clothing. Rinse skin with water or shower
P304+P340	IF INHALED: move person to fresh air and keep comfortable for breathing
P305+P351+P338	IF IN EYES: rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing
P310	Immediately call a POISON CENTER/doctor/...
P321	Specific treatment (see ... on this label)
P330	Rinse mouth
P333+P313	If skin irritation or rash occurs: get medical advice/attention
P342+P311	If experiencing respiratory symptoms: call a POISON CENTER/doctor/...
P363	Wash contaminated clothing before reuse

Storage

P405 Store locked up

Disposal

P501 Dispose of contents/container in accordance with local regulations

SYMBOLS
USED

Manufactured by

Distributed by

Product code

Batch number

Use before
Month & Year

Do not reuse
single use only

Notified body
identification
number

Corrosive

Serious health
effects

Harmful/Irritant

Do not release into
the environment
after use

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